

How to stream sounds from your laptop to hearing aids using TV streamer (stereo to stereo cable)



Step 1

Step 2

Step 3

Step 4

Step 5

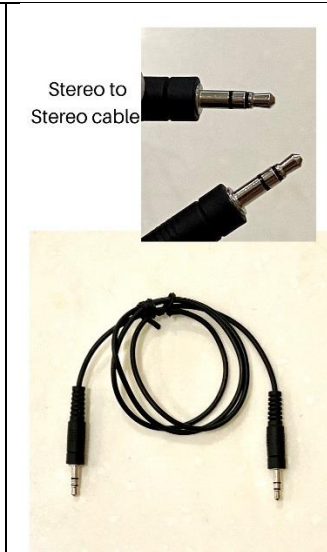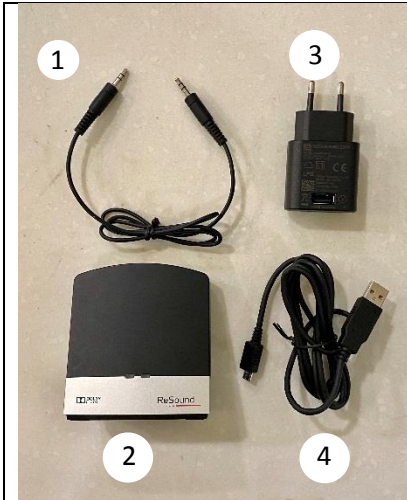

You can use the stereo-to-stereo cable to connect the TV streamer to the laptop.

These are the items you need:

- 1) Stereo-to-stereo cable
- 2) TV streamer
- 3) Power plug
- 4) Cable for power plug

The stereo-to-stereo cable has two stereo ports on each end

Connect one of the stereo ports of the stereo-to-stereo cable to the TV streamer's stereo port (as indicated by the arrow)

Connect the other end of the stereo-to-stereo cable to your laptop as per the above picture.

This is also known as the 3.5mm headphone jack

You need to power up the TV streamer next. You can either connect to the laptop using the usb cable marked as (4) or use the 2-pin power plug marked under (3) outlined in step 1 to connect to the wall's pocket socket

The TV streamer will light up and ready to stream sounds to your hearing aids once this is done successfully.

Enjoy the new listening experience!